

SCRAM Remote Breath Test to Zero Feature



Purpose

Use this quick sheet as a reference when utilizing the Remote Breath “Test to Zero” feature.

Test to Zero Parameters

- Once AMS enables the “Test to Zero” feature, all Remote Breath clients will be required to continue to take breath tests until the client’s breath sample is below the established Breath Alcohol Concentration (BrAC) threshold.
- When SCRAMNET® receives a failed Remote Breath test, the client will continue to receive on-demand test prompts until SCRAMNET receives a passed test.

Alert	Result	BrAC	Image Review	Event Date/Time	Received At	Note	Status
⚠	Missed On Demand	--		10/31/2017 @ 8:43 PM	10/31/2017 @ 8:43 PM		NEW
⚠	Failed On Demand	0.073		10/31/2017 @ 8:03 PM	10/31/2017 @ 8:03 PM		NEW
⚠	Failed On Demand	0.080		10/31/2017 @ 7:42 PM	10/31/2017 @ 7:43 PM		NEW
⚠	Failed Client Initiated	0.087		10/31/2017 @ 7:22 PM	10/31/2017 @ 7:22 PM		NEW
	Passed	0.000		10/31/2017 @ 11:32 AM	10/31/2017 @ 11:33 AM		NONE
⚠	Failed On Demand	0.113		10/31/2017 @ 11:18 AM	10/31/2017 @ 11:19 AM		NEW
⚠	Failed	0.136		10/31/2017 @ 11:01 AM	10/31/2017 @ 11:02 AM		NEW

SCRAM Remote Breath Test to Zero Feature

- If the client does NOT take the on-demand breath test, no additional prompts will be sent, and a Missed Test alert will be generated.

The screenshot shows the SCRAMNET web interface. At the top, there is a navigation bar with icons for Home, Inventory, Search, and Admin. Below this is a breadcrumb trail: Call Center Supervisor / Agency: Trevor UAT EM / Agent: Testing, G's / Client: Smith, John T. The main content area is titled "RB Results" and includes a "Filter Results" section with a "Date Range" dropdown set to "Last 7 Days" and "Event Categories" checkboxes for Communication, Enrollment, Power/Battery, Tamper, and Test Results. Below the filters is a "Results" table with columns: Alert, Result, BRAC, Image Review, Event Date/Time, Received/E, Note, and Status. The table contains four rows of data, with the first row highlighted by an orange border.

Alert	Result	BRAC	Image Review	Event Date/Time	Received/E	Note	Status
Missed	On Demand	--		10/31/2017 @ 8:43 PM	10/31/2017 @ 8:43 PM		NEW
Failed	On Demand	0.073		10/31/2017 @ 8:03 PM	10/31/2017 @ 8:03 PM		NEW
Failed	On Demand	0.080		10/31/2017 @ 7:42 PM	10/31/2017 @ 7:43 PM		NEW
Failed	Client Initiated	0.087		10/31/2017 @ 7:22 PM	10/31/2017 @ 7:22 PM		NEW

- A courtesy reminder for an on-demand test will be sent to the client if the client is enabled for courtesy reminders.
- If multiple failed breath tests are uploaded from the Remote Breath device in a single communication, only one on-demand test will be sent to the client.
- If SCRAMNET does not communicate with the Remote Breath device for two hours, any pending on-demand tests will expire. Reasons for no communication include:
 - Loss of cellular service
 - Dead battery
- If a failed breath test is not received in 24 hours, SCRAMNET will no longer request an on-demand test.